

# Quick User Guide Yuwell 7F-10 Oxygen Concentrator

- 1) Place the unit 2 feet from the wall (all barriers) near an electrical outlet.
  - 2) Check the air inlet/course filters on the sides of the concentrator. Ensure these are clean.
  - 3) Add distilled water in the humidifier bottle. [*between the Maximum and Minimum marks*]
  - 4) Connect oxygen accessories such as a humidifying bottle, nasal cannula/face mask, and/or extension tubing to the oxygen outlet of the humidifier bottle. [*See image below*]
- NOTE: Ensure that the second oxygen outlet is closed at all times. [IF NOT IN USE] – See image aside.**
- 5) Insert the 3 pin power plug from the AVS volt guard/ surge protector in the wall socket. Ensure that the switch is turned on.

**For emergency purposes only**

➤ **Press the yellow “BYPASS” button on the AVS to power the device immediately.**

- 6) Locate the ON/OFF “0/1” switch on the front of the unit, and turn it to the “1” position (on). The I/O /running indicator lights “Green”.
- 7) Adjust the flow meter knob as prescribed by the doctor/according to patient needs. Wait for 3-5 minutes for the oxygen concentration to stabilise. Oxygen indicator turns “Green”.

**NOTE: When the oxygen indicator lights “Red” – DON’T USE THE DEVICE.**

- 8) Connect the cannula to the patient and offer therapy for the time specified by the doctor or until the patient saturation is normal.
- 9) At the end of treatment, switch off device and disconnect accessories. Pour distilled water and clean concentrator using a moist towel.
- 10) Once a week, record the hours of use on the device and percentage oxygen purity on the template provided.

